

IT HELPS ME EAT, TALK & SMILE



GALEN

A guide for patients who have been prescribed **GelX[®] Oral spray**

for both the treatment & prevention of Oral Mucositis



GelX[®] Oral Spray has a mechanical action indicated for the management of pain and relief of pain by adhering to the mucosal surface of the mouth, soothing the lesions caused by chemotherapy and/or radiation therapy.

It can also be used as a preventive treatment before chemotherapy and/or radiation therapy to protect the oral mucosa. Alcohol free.

gelX[®]
ORAL SPRAY

gelxuk.com

What does Oral Mucositis feel like?



Difficulty talking



Soreness



Trouble eating



Burning sensation

Directions for Use¹



Step 1

Take the top off and apply vaporizer.




Step 2

Direct the vaporizer towards the lesion inside the mouth and spray three times.



Step 3

Repeat three times per day. Avoid swallowing any substances, solid or liquid, for at least one hour after each application.

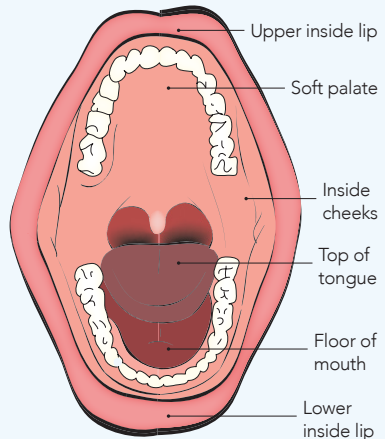

Adverse incidents should be reported. Reporting forms and information can be found at www.mhra.gov.uk/yellowcard.

Adverse incidents should also be reported to Galen Limited on 028 3833 4974 and select the customer services option, or e-mail customer.services@galen-pharma.com. Medical information enquiries should also be directed to Galen Limited.

Reference

1. Gel X Oral Spray Instructions for use

GELX is a registered trademark of Sunstar Suisse SA.

MAT-GELX OS-UKI-000002 Date of Preparation: April 2022

GALEN